Wonderland 102

Norge:	Regulerbar seng
Sverige:	Reglerbar säng
Danmark:	Regulerbar seng
Suomi:	Säätösänky
Nederland:	Verstelbaar bed
Great Britain:	Adjustable bed
Deutschland:	Das justierbare Bett

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Before operating the bed


GB

102 adjustable slat frame is designed to be used in a bed frame. You need to make sure that the wooden lists at the inner side of the bed frame are of high quality. The lists must be wide enough to keep the bed frame from falling down when operated. Rise and lower the slat frame a few times before placing the mattress.

Emergency power battery

The bed is delivered fully assembled, and the battery should already be connected to the motor. The batteries have a normal life expectancy of one year, after which time they must be replaced. To change the batteries, do as follows: (See illustration.)

1. Remove the seal on the battery, if necessary.
2. Attach clamps to batteries.
3. Insert batteries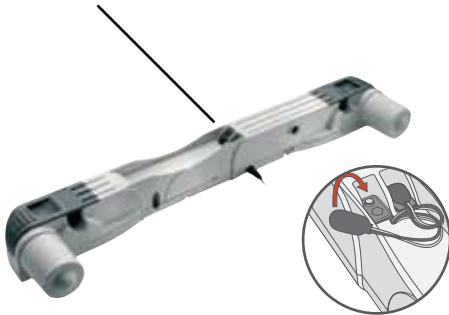

Remove the shipping safety


For a safer transport, the slat frame's top and bottom are fastened with a plastic ribbon. When the slat frame is placed in the bed, these can be removed.

Power

Place the plug in a 220V socket. There will be no electricity to the bed before you activate the remote control.

NB!

Do not sit or stand on the top or foot part of the bed when it's not in horizontal position.

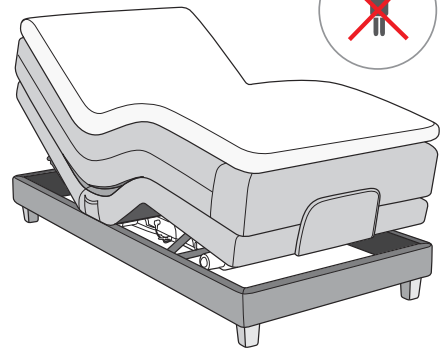Remote control

The remote control operates by Radio frequency. Radio frequency has a long range. The receiver is integrated in the motor.

By using the remote control, you can rise or lower the top or foot part of the bed to the desired position with the remote control. You can rise or lower both simultaneously.

The remote control has built-in batteries, and must be changed when the batteries are discharged.

Facts

GB

Power switch

102 has an electricity circuit breaker in the plug, which ensures that there is no supply of electricity to the motor when the bed is not operated.

Emergency lowering in case of power failure

The bed has an emergency power battery. In case of power failure, you can still lower the bed to horizontal position.

Maintenance

To make sure this adjustable slat frame operates as expected as long as possible, some maintenance in addition to normal cleaning is needed.

Occasionally, there might be a need for oiling the adjustable metal fittings with thin lubricating oil. Lubricating oils in a spray tin might be used.

If there should be any noise coming from the slat frame, try to tighten the screws. Wood will naturally change in different temperatures, and the screws might thus need tightening.


Adjusting the firmness

By sliding the black plastic fittings outwards or inwards, you can adjust the firmness under the hip area. Try different positions till you find the firmness that fits you.

Changing the motor

Your new 102 slat frame is designed to last for many years. If, in spite of this, a problem should arise, most parts are easy to replace. This includes the main motor.

Remove the two plastic covers holding the motor in place (see illustration). The motor is now released. Place the new motor in the correct position and slide the plastic covers back into locked position.

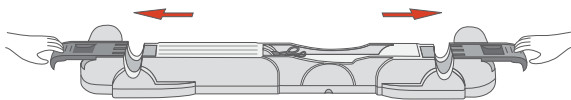Problems?

GB

If the bed does not work:

Should problems arise related to operating your 102 slat frame; the following check list can assist you in finding the reason/solution.

Problem:

The bed will not be risen or lowered

Cause:

- Is the plug in place?
- Is there electricity in the socket?
- Are the batteries connected to the emergency power battery?
- Are the batteries in the emergency power function defect?
- Remove the wall plug and wait two minutes before placing it back in.
- Has the remote control flat batteries? The remote control must be changed when the batteries are discharged.

Problem:

The bed will not lower on emergency power

Cause:

Remove the wall plug. The emergency power function only works if there is no supply of electricity to the bed. If the bed still does not work, it might indicate an error in the motor or the remote control. If you have two beds, this can be tested by using the other bed's remote control. If the bed works with the other one, this indicates an error in the original one. If the bed still does not work, this indicates an error in the motor, then contact your dealer.

WONDERLAND AS

Øran Vest, N-6300 Åndalsnes

Tlf:+47 71 22 78 00, Fax:+47 71 22 78 01

www.wonderland.no

wonderland@wonderland.no